


HOW TO APPLY TAGFASTER TAGS




- Tag from the front with the tag facing forward in the ear
- Only apply TagFaster tags with the TagFaster Applicator
- Correct application ensures better tag retention
- We recommend that you dip the tag pins in antiseptic before applying the tags

LOADING

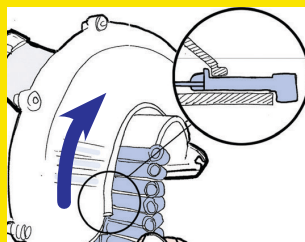


1. Insert tag strip with pins and tag ridge covered by feed horn.

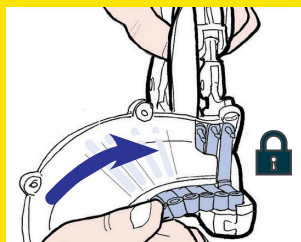


2. Push tags until they "stop" and lock in place.

Tag from the front with the tag facing forward in the ear.


TAGGING

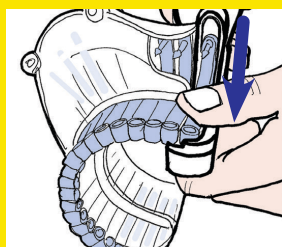


1. Press down on tag to align it within the hammer.

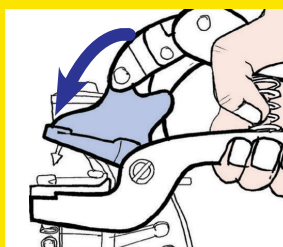

2. Squeeze applicator halfway to prime for tagging.

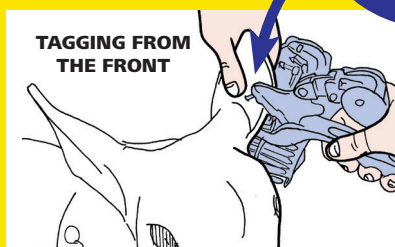


3. Ensure the tag is positioned in the ear as close to the head as possible avoiding the main blood vessels and ridges of cartilage. In case of young animals, allow for ear growth by leaving the tag to stick out by 1/4 of its length from the ear. Apply the tag by holding the animal securely and flattening the animal's ear. TagFaster tags and applicator are suitable for use in marking cradles.

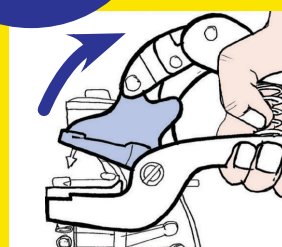


4. Fully and sharply release the handle to feed next tag.

TROUBLESHOOTING

UNLOADING

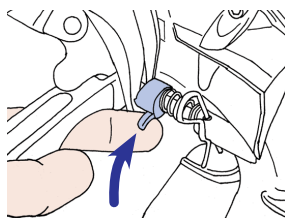


1. Change tag strip (if required), pull tab up and slide tags out.

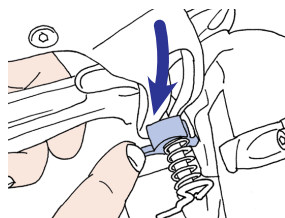


2. Return tab to original position.


CHANGING THE BLADE

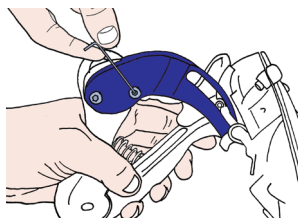


1. Use the Allen key provided to remove the 2 screws holding the tooth in place.

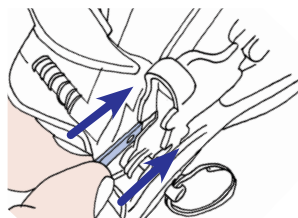


4. Slide the blade into place between the grooves into the hammer.

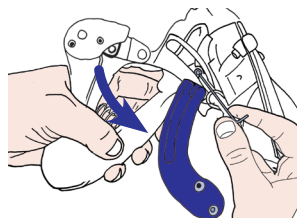


2. Swing the tooth down to access the blade. Unscrew with the Allen key provided.

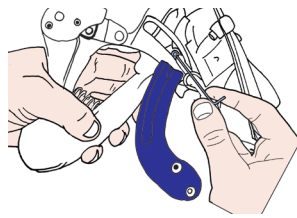

5. Tighten the blade screw using the Allen key.

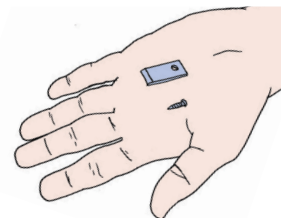

3. Take the new blade and screw provided.

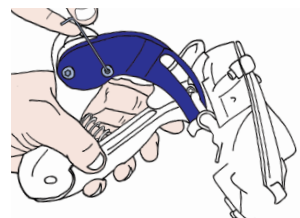

6. Reposition the tooth and tighten.